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

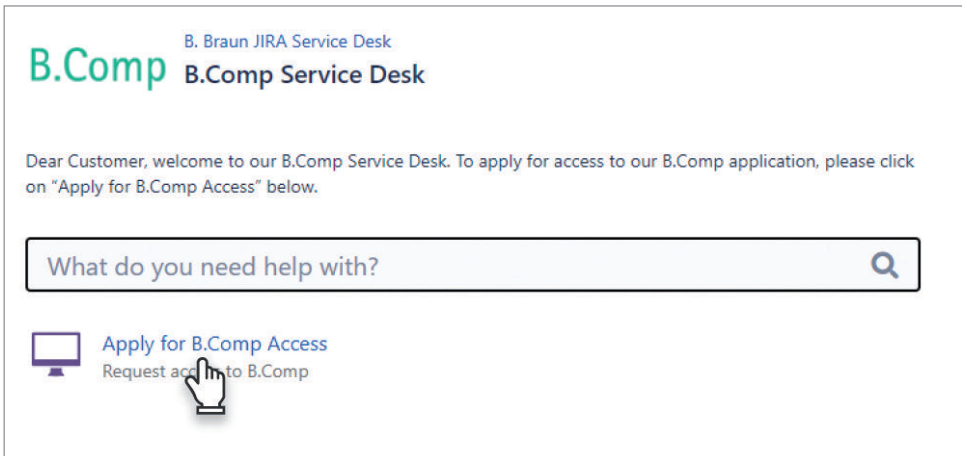
The screenshot shows the B. Braun JIRA Service Desk interface. At the top left, the B.Comp logo is displayed in green, followed by the text "B. Braun JIRA Service Desk" and "B.Comp Service Desk". Below this, a message reads: "Dear Customer, welcome to our B.Comp Service Desk. To apply for access to our B.Comp application, please click on 'Apply for B.Comp Access' below." A search bar with the placeholder text "What do you need help with?" and a magnifying glass icon is positioned below the message. Underneath the search bar, there is a button with a computer monitor icon, the text "Apply for B.Comp Access", and the subtext "Request access to B.Comp". A hand cursor is shown clicking on the button.

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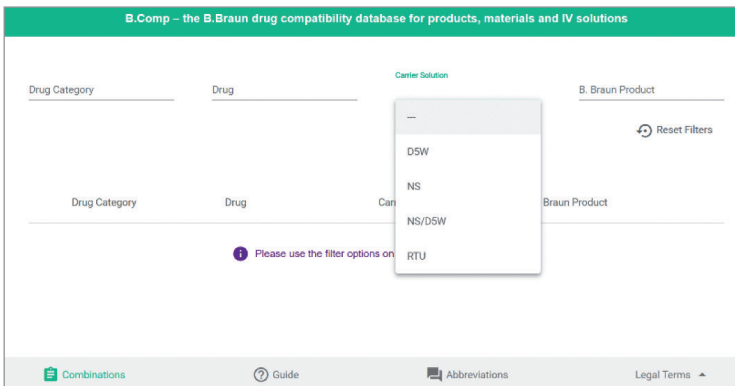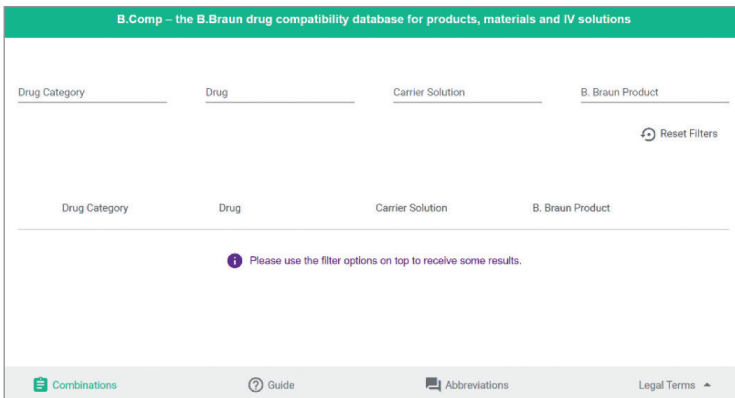B.Comp – the B.Braun drug compatibility database for products, materials and IV solutions

Drug Category: Chemotherapeutic ✕

Drug: Paclitaxel ✕

Carrier Solution: NS/D5W

B. Braun Product: Easypump II ST/LT ✕

Reset Filters

Drug Category	Drug	Carrier Solution	B. Braun Product
✓ Chemotherapeutic	Paclitaxel	NS/D5W	Easypump II ST/LT

Combinations Guide Abbreviations Legal Terms

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< Back B.Comp – the B.Braun drug compatibility database for products, materials and IV solutions

Combination

 Easypump II ST/LT

Drug Category: Chemotherapeutic

Drug: Paclitaxel

Carrier Solution: NS/D5W

Drug Stability / Compatibility List

Concentration (Range)	Refrigerated (2°C - 8°C)	Room Temperature (15°C - 25°C)	Storage Condition	Test Summary
0.3 mg/ml	7 Days	24 Hours	Combined	

Combinations Guide Abbreviations Legal Terms

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The screenshot shows a web application interface for 'B.Comp - the B. Braun drug compatibility database for products, materials and IV solutions'. It features a 'Selected Drug Stability / Compatibility List' and a 'Lab Reports' section.

Drug	Concentration (Range)	Carrier Solution	Refrigerated (2°C - 8°C)	Room Temperature (15°C - 25°C)	Storage Condition
Paclitaxel	0.3 mg/ml	NS/D5W	7 Days	24 Hours	Combined

Title	Test Facility	Release Date	Download
Paclitaxel	Ecotox	07/09/2016	
Paclitaxel	Ecotox	06/09/2016	

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The screenshot shows the search interface of the B.Comp database. It includes filter options for 'Drug Category', 'Drug', 'Carrier Solution', and 'B. Braun Product'. A 'Reset Filters' button is visible on the right side. Below the filters, a message states: 'Please use the filter options on top to receive some results.'

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B. Braun Deutschland GmbH & Co. KG